



VariVac™\*  
Wound Drainage System

Adjustable  
Constant and  
Gravity Settings  
for Consistent  
Control



# VariVac™\* gives you the control that you need in a wound drainage system

## EASY TO ADJUST VACUUM

Designed for all surgical cases, simply dial the amount of vacuum desired from approximately 0 kPa to 15 kPa.

## CONSTANT VACUUM

Will maintain the vacuum you require to its capacity.

## GRAVITY / PASSIVE SETTING

This unique feature allows you to not only minimise excessive drainage but also return to normal drainage when required by simply adjusting the dial.

## CRYSTAL CLEAR

Reservoir container allows for easy identification of exudate flow.

## PRE-VACUUMED AND RECHARGEABLE

For ease of use and convenience.

## INFECTION CONTROL

Many features include one-way valve, quick connectors, strong polycarb construction for maximum safety and protection.

## TOTALLY CLOSED VACUUM SYSTEM

Safe containment of exudate eliminates aerosol spray.

## VACUUM INDICATOR

Large, easily visible for identification of vacuum present in reservoir.


## AUSTRALIAN DESIGNED

Designed by an Australian company. Australian Patent Reg. No. 36448/95



### The innovation

Gravity back to normal drainage by just turning the dial.

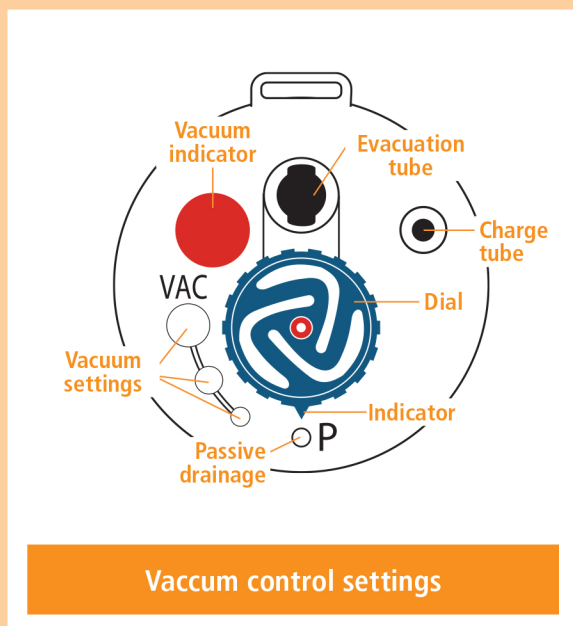

Vacuum control settings

### Vacuum adjustment

To regulate the vacuum level, simply turn the vacuum dial.


Range is adjustable between the approximate settings below:

Passive (Gravity)	0 kPa
Low	5 kPa (37 mmHg) / 51 cmH <sub>2</sub> O
Medium	10 kPa (75 mmHg) / 102 cmH <sub>2</sub> O
High	15 kPa (113 mmHg) / 153 cmH <sub>2</sub> O

## VariVac™\*

### Australian made wound drainage system

Product Code	Description	Qty
VC 500-02	10 Fr (3.2 mm, 1/8") kit complete	20
VC 500-03	15 Fr (4.8 mm, 3/16") kit complete	20
VC 500-04	19 Fr (6.4 mm, 1/4") kit complete	20
VC 500-11	500 cc Reservoir kit Y connector	20
VC 500-12	500 cc VariVac™ reservoir only	20
VC 501-22	Varivaca manual pump with filter	Each
VC500-61	Varivac™ Combo with VariBlake 10Fr with Trocar and connecting tubing with connector	20
VC500-62	Varivac™ Combo with VariBlake 14Fr with Trocar and connecting tubing with connector	20
VC500-63ST	Varivac™ Combo with VariBlake 19Fr with Trocar and connecting tubing with connector	20
VC500-64	Varivac™ Combo with VariBlake 8mm Flat Drain with trocar	20
VC500-65	Varivac™ Combo with VariBlake 11mm Flat Drain with trocar	20

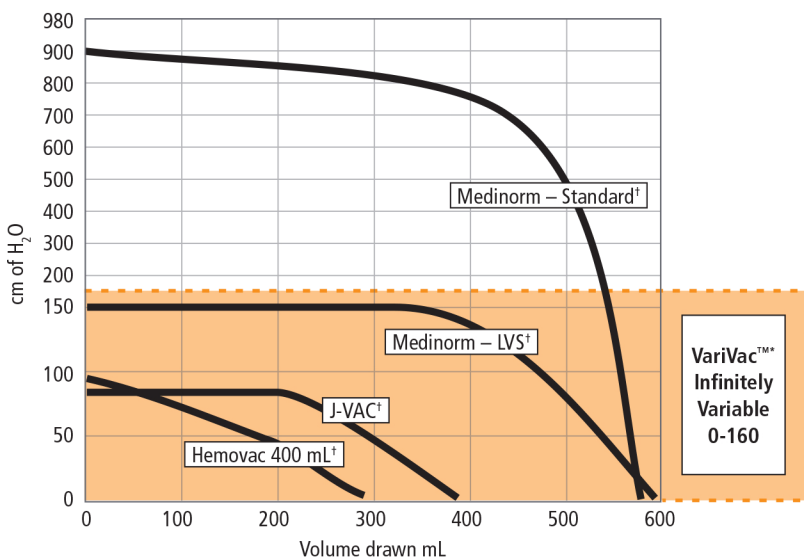

† Information drawn from company's promotional literature.